

USER INSTRUCTIONS – PEAK VISOR (Patent Pending No. GB2011275.1)

PRODUCT DESCRIPTION

- This model is developed to incorporate an existing cap or peak visor
- It is a fashionable solution, while offering protection from aerosol infectious diseases/virus by creating a physical barrier for the user to protect the whole face, eyes, mouth and nose
- This model is particularly adaptable by companies to integrate into their existing uniform or by any one wishing to use with their own cap
- ViSEPP™ Peak Visor is easy to assemble



USER INSTRUCTIONS

To Wear

- Place Cap or Peak upside down, and slot the peak of the Cap through the cut-out slot in the Visor whilst keep the Visor flat
- Pull the Visor to the edge of the Cap (position as shown in the picture)



- From the position in picture one pull the Visor up towards you so that it is now pointing upwards
- Ensure the slots near each tab are aligned with the edge of the Peak on the Cap



- Insert the Visor securing tab inside the Cap and fold the rear tab behind the lining band on the cap or visor



- Straighten any areas so the Visor takes shape



- Your Peak visor and Cap are now ready to wear
- To remove the Visor from the Cap, just pull the Visor off the Peak of the Cap



Cleaning

- Wipe visor with alcohol wipes or with warm soapy water
- Allow foam to dry before use if it becomes wet during cleaning
- Dry visor with a lint free cloth

KEY FEATURES

- Patent Pending
- Lightest Visor on the market (not including cap)
- Extremely comfortable to wear even after hours of use
- 100% recyclable
- Compatible with Facial recognition technology
- Re-usable
- Easy to Clean
- Designed and manufactured in the UK
- Can be fitted onto existing cap or peak cap (Gold Cap)
- Unobtrusive to the user and the public
- Excellent for use with corporate branding
- Optically clear, anti-glare, medical grade PET screen
- No materials cause skin irritation
- Easy to assemble and remove
- Delivered straight through your letterbox, no need to stay in for delivery

COMPONENTS AND SPECIFICATIONS

- Visor: 180 micron medical grade PET acetate, size: width 340mm X depth 245mm - Recyclable
- Packaging Box - Recycled cardboard
- Protective Tissue - Recyclable
- Delivery bag - Biodegradable